




Infusomat® Space


Loading and Unloading Procedure

Loading Procedure

- 1 Turn the pump on by pressing .
Press  and  to open the pump door.
- 2 **Always start at the right hand side.** Insert the two holed clip.




- 3 Insert the second white clip. **Ensure that the silicone tubing is not twisted.**

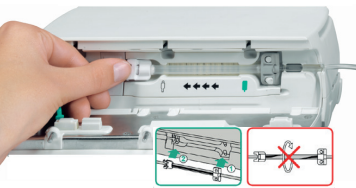


- 4 Ensure the green opening lever is pressed down and the yellow hazard light is flashing.



- 5 Insert the green clamp into the anti free-flow slot. **When correctly inserted the yellow hazard light will stop flashing.**

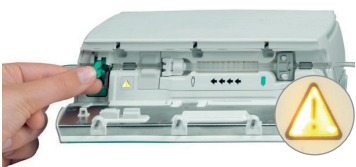




- 6 Push and hold the door firmly for a few seconds to close. Confirm giving set with .

SCAN THIS QR CODE TO WATCH A VIDEO ON THE CORRECT LOADING PROCEDURE




Unloading Procedure

- 1 Press  to stop the infusion. The green LED goes out.
Close the roller clamp.


- 2 Press . Confirm the prompt with .



- 3 Pull **firmly** down on the door to open and engage the anti-freeflow.

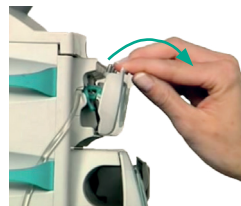

- 4 Press down the green opening lever completely until it locks in place.

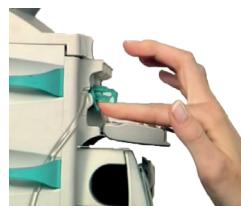

- 5 Remove the line and close the pump door.

SCAN THIS QR CODE TO WATCH A VIDEO ON THE CORRECT UNLOADING PROCEDURE



This poster is provided for reference only. Prior to operating this equipment all users should consult the full Instructions For Use.